

Instructions for Use

ClariCare Eye Drops

Eye drops with sodium hyaluronate 0.15% and dexpanthenol 2%

DESCRIPTION

ClariCare Eye Drops care for and moisturise the ocular surface and alleviate the symptoms of Dry Eye Syndrome such as irritated, red, itchy, watery and/or very sensitive ocular surface by forming a preservative-free, viscoelastic, clear protective film on the cornea without impairing visual performance. This is achieved by the high water-binding capacity of the sodium hyaluronate contained, the sodium salt of hyaluronic acid, which also occurs naturally in the vitreous body of the eye, among other things.

In addition, **ClariCare Eye Drops** contain provitamin B5 (Dexpanthenol), which has the ability to bind a high quantity of water, moisturises and cares for the eye, on the one hand, and on the other hand, supports the Sodium Hyaluronate to hold moisture.

As a result, **ClariCare Eye Drops** provides a long-lasting relief, moisturising and care from discomforts associated with:

- mechanical stress, e.g. during wearing of hard or soft contact lenses
- environmental stress, e.g. induced by air conditioning, wind, coldness, dryness or air polluted by cigarette smoke for example
- strained eyes, e.g. from working on the computer, with microscopes and during/after long car journeys
- a foreign body sensation in the eye
- regular use of certain medicines
- decreasing tear production with age
- allergens such as pollen, animal hair, dust, etc.
- dry eye as a result of hormonal disorders e.g. in menopause
- diagnostic or surgical interventions on the eye for pre- and post-operative care

Preservative-free **ClariCare Eye Drops** are also well tolerated during long-term use.


ClariCare Eye Drops are suitable for contact lenses wearers. The contact lenses do not have to be removed before/during the application of the eye drops.

APPLICATION

To open the bottle, remove the tamper-evident closure by pulling on the designed flap until it is completely removed (1). Then remove the cap by pulling upwards (2). Tilt your head back a little, look upwards and gently pull the lower eyelid away from the eye. Hold the bottle with the dropper opening vertically downwards and instill 1 - 2 drops 3 - 5 times a day into the conjunctival sac of the eye (3 + 4) by applying evenly strong, prolonged pressure. Close your eyes slowly to allow an even distribution of the liquid on the eye surface.

Due to the bottle's special sterile air filter system, a few seconds must elapse after each use without pressure on the bottle so that sufficient air can flow in again. Without this waiting period, further use of the device is only possible with difficulty.

Avoid any contact of the opening with the eye or the hands. A remaining drop on the opening should be removed by light tapping on the bottom of the bottle. Close the bottle now by pressing slightly the cap on the opening (5).

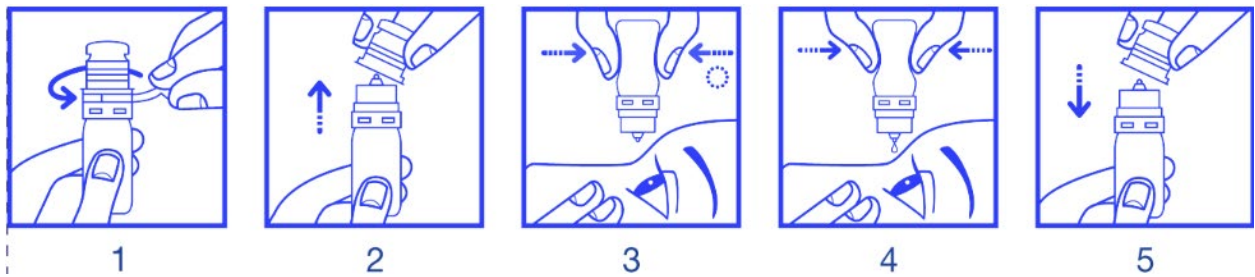

The more the bottle is emptied, the more pressure is required to remove the drops as a result of physical properties.

If the discomfort gets worse or does not improve, please consult your ophthalmologist.

COMPOSITION

Sodium hyaluronate 0.15%, dexpanthenol 2%, sodium chloride, sodium citrate, citric acid, water for injections.

PACKAGING SIZE

1x10 ml

PRECAUTIONS AND WARNINGS

- Do not use the product if the tamper-evident closure of the cap is broken or the bottle is damaged.
- Do not touch the eye with the dropper.
- Do not use if you are allergic to any of the ingredients.
- In very rare cases, the use of the eye drops may cause irritation in very sensitive eyes or eye lids. The following eye disorders may occur: redness, pain, burning, stinging, itching, swelling, foreign body sensation.
- If you also use other eye drops / eye ointment, wait about 15 minutes. **ClariCare Eye Drops** should always be used last.
- Do not use in infected eyes.
- Temporary blurring for a brief period may be experienced after application; avoid driving, operating machinery and wait until normal vision is restored.
- Do not use the eye drops beyond the expiry date which you can find on each multi-dose container and on the folding box.
- After first opening use not longer than 12 months.
- Due to the special innovative sterile filter system of the dropper insert a small quantity always remains in the bottle. To ensure unrestricted dropping capability, regular use is recommended.
- Please keep the product out of reach of children.
- In case a serious incident has occurred during the use of this medical device or as a result of its use, please report it to the manufacturer and the competent authority.

Storage

Store at 2 °C to 25 °C. Do not freeze. Close the bottle after using with the cap. Protect from light by storing the bottle in the original folding box.



Penta Arzneimittel GmbH
Werksstraße 3
92551 Stulln
Germany



Bayer (Pty) Ltd
Collaboration Hub, 1st Floor
9 Country Estate Drive, Midrand
South Africa
Tel.: +27 11 921 5000
Co. Reg. #: 1968/011192/07

CE 0483

Date of revision: November 2025

EXPLANATION OF THE ICONS

	Batch code
	Expiry date
	Sterilized using aseptic processing techniques
	Single sterile barrier system with protective packaging outside
	Single sterile barrier system
	Consult instructions for use
	Temperature limit and storage temperature
	Keep away from sunlight, store the single dose container protected from light in the original folding box.

	Do not use the eye drops for longer than 12 months after opening for the first time
	Do not use if package is damaged (single dose container/pouch) and consult instructions for use
	Medical device
	Manufacturer
	Date of manufacture
	Distributor
	CE marking = symbol of free marketability in the EU with indication of the involved Notified Body
	Unique device identifier